

Hjertevakten®

En kamp for livet - mot klokken

How to connect ShockLink to train with a ZOLL defibrillator

Below are instructions to help you connect ShockLink to the following ZOLL X Series, Propaq Series, AED Pro, AED Plus, and R Series defibrillators.

ZOLL X and Propaq MD Series

A [ZOLL electrode adaptor](#) is needed to insert ShockLink into a ZOLL defibrillator cable.

In this set up, ShockLink features can be used in the normal way and the full feature set can be utilised when Resusci Anne QCPR and SimPad SkillReporter is used.



Hjertevakten®

En kamp for livet - mot klokken

ZOLL AED Pro and AED Plus

Start by inserting ZOLL training pads into the defibrillator and ShockLink into the plug on the ZOLL training pad's lead, using the [ZOLL electrode adaptor](#). As shown in the image to the right.

[To find ZOLL training pads you can use ZOLL's electrode finder >](#)

We have used ZOLL CPR-D-padz Demo pads in our example set up.



You can now choose which training pads you would actually like to use on the manikin.

You can either:

a. Use Laerdal training pads

By choosing to use Laerdal training pads you will be able to use ShockLink features in the normal way and the full feature set can be utilised when Resusci Anne QCPR and SimPad SkillReporter is used.

The Laerdal training pads need to be connected to the ShockLink that was inserted into the ZOLL training pads.

As shown in the image below, when connecting ShockLink to the ZOLL CPR-D-padz Demo pads, the electrodes can

b. Use ZOLL training pads

If you choose to use the ZOLL training pads, correct pad placement can't be detected and such events will not be logged to SimPad SkillReporter.

ShockLink will allow for simulated heart rhythms to be displayed on the monitor and for an instructor to prompt changes to these rhythms, by using the remote or the app.

No training pads need to be inserted into ShockLink in this set

Hjertevakten®

En kamp for livet - mot klokken

be detached from the compression measurement sensor.



up, as shown below.



ZOLL R Series

To connect ShockLink to this defibrillator, you can use the same method as outlined above for the ZOLL AED Pro and AED Plus defibrillators but, if ZOLL training pads with a green connector are used, an additional ZOLL connector is required to insert them into the defibrillator.

[To find ZOLL training pads, and required ZOLL connectors, you can use ZOLL's electrode finder >](#)

